

USER MANUAL



Proride
e - M o b i l i t y

Ultralight Pushchair

How to fold your wheelchair

STEP 1 Fold both armrest all the way down.



STEP 2 Pull the quick release handle and slightly press down the seat.



STEP 3 Lay down the wheelchair as shown in figure and gently release the handle bar. Ensure the tyres are folded inwards.



STEP 4 Wheelchair is now safely folded. It can be fit in and out of small car's trunk.



How to unfold your wheelchair

STEP 1 Folded position.



STEP 2 Lift up the handle bar while stepping on the back frame between the tyres.



STEP 3 Unfold both armrest and footrest.



STEP 4 Wheelchair is now unfolded.

